Manufacturer: Jiangsu Weibang Medical Products Co., Ltd. Model Tested: KN95 Diposable Particulate Respirator Date Tested: May 16, 2020

These findings pertain to the Jiangsu Weibang Medical Products Co., Ltd., KN95 Diposable Particulate Respirator. The packaging for this product indicates that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

Ten respirators were submitted for evaluation. The samples were tested using a modified version of NIOSH Standard Test Procedure (STP) TEB-APR-STP-0059. This modified assessment plan can be found <u>here</u>.

No certificate of approval was provided with the samples received; therefore, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 99.65% and 97.89%, respectively. All ten respirators measured more than 95%.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

In addition, this product is an ear loop design. Currently, there are no NIOSH-approved products with ear loops; NIOSH-approved N95s have head bands. Furthermore, limited assessment of ear loop designs, indicate difficulty achieving a proper fit. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirator's represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for <u>Crisis Capacity Strategies (during known</u> <u>shortages)</u>.

Evaluation of International Respirators

Test: Modified TEB-APR-STP-0059

Date Tested: May 16, 2020

Report Prepared: May 17, 2020

Manufacturer: Jiangsu Weibang Medical Products Co., Ltd.

Item Tested: KN95 Diposable Particulate Respirator

Country of Certification: China (GB2626-2006)

Filter	Flow Rate (Lpm)	Initial Filter Resistance (mmH ₂ O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency
1	85	8.8	1.22	1.59	98.41
2	85	10.1	0.34	0.35	99.65
3	85	9.9	0.59	0.62	99.38
4	85	8.9	0.44	0.49	99.51
5	85	10.5	1.07	1.09	98.91
6	85	10.0	0.77	0.79	99.21
7	85	9.5	1.81	2.11	97.89
8	85	10.8	0.59	0.59	99.41
9	85	13.7	0.49	0.49	99.51
10	85	9.2	1.05	1.24	98.76
Minimum Filter Efficiency: 97.89			Maximum Filter Efficiency: 99.65		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.

NPPTL National Personal Protective Technology Laboratory

Pictures have been added to the end of this report.



Instruction

KN95 Disposable Particulate Respirator Size: Adult Material: PP Non-woven, Melt-blown, Cotton Non-woven Composition: 5 layers of non-woven, elastic earloop and plastic nose clip. Intended Use: Wear for non-invasive operation as barrier to provent direct pasage of pathogens, microorganisms, particles, etc. Validity Period: 2 Years Production Date & Batch Number: See sealing of package Excutive Standard: GB2626-2006 Manfucaturer: Jiangsu Weibang Medical Products Co. , Ltd. Address: 282-2621, Changjiang North road, Xinwu district Wuxi

Warning:

Do not use if package is damaged.

Designed for single use, do not reuse

Dispose properly after use in case of cross pollution







Wearing Guide





1. Remove package, unfold the mask, and face the mask with nose clip upward.

2. Pull the rubber bands from edge by fingers, then hang the bands around ear back.

Make sure mouth and nose
are covered properly, then
adjust mask to your coziness.
Press nose clip tight onto nose.
Gradually move index fingers
to both sides, to make sure mask
pressed tight on face.











