

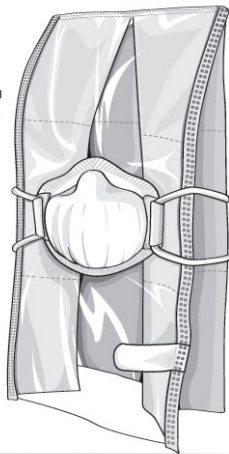
Champak

Disposable Particulate Respirator With Protective Hood

- Easy Breathing
- Long Duration
- High Filtering Efficiency



Champak Enterprise Co., Ltd
No. 27-1 Jhaiming St. Dasi Town
Taoyuan County 335 Taiwan
+886-3-380-8818



THIS RESPIRATOR IS APPROVED ONLY IN THE FOLLOWING CONFIGURATION:

TC-	Protection ¹	Respirator	Cautions and Limitations ²
		A520H	
84A-XXXX	N95	X	ABCJMNOP

1. Protection
N95- Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restriction may apply.
2. Cautions and Limitations
A - Not for use in atmospheres containing less than 19.5 percent oxygen.
B - Not for use in atmospheres immediately dangerous to life or health.
C - Do not exceed maximum use concentrations established by regulatory standards.
J - Failure to properly use and maintain this product could result in injury or death.
M - All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
N - Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
O - Refer to User's Instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
P - NIOSH does not evaluate respirators for use as surgical masks.

Storage and Shelf life

The respirator should be placed in a container and stored in a clean cool and dry atmosphere to ensure that it is protected against dust, chemicals, moisture, excessive heat and direct sunlight.

Instructions for use

1. This respirator must not be used for more than single shift (8 hours). Always discard and replace the respirator when the breathing resistance becomes excessive or at the end of every working shift.
2. No alteration or modification is allowed. Never use damaged respirators, in that case discard these respirators.
3. The wearer of the respirator must be well trained to use properly before starting work. Facial hair (beard or sideburns) of the user within the area of the sealing edges of the respirator may jeopardize the protection efficiency of the respirator itself.
4. Before use of these respirators, a written respiratory protection program must be implemented, meeting all the requirements of OSHA 29 CFR 1910.134, including training, medical evaluation and fit testing.

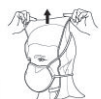


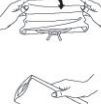

Warning

1. Do not use this respirator as protection against toxic particles, gases or vapors, solvents from spray-painting operations, or in explosive atmospheres.
2. Do not use this respirator in tanks, enclosed places or in any circumstances where gas is likely to be present or in oxygen deficient atmospheres.
3. This respirator should not be worn when conditions prevent a good face to face piece seal. Such conditions includes the growth of beards or mustache and sideburns and the use of spectacles or any other devices containing temples or straps which pass between the respirator sealing edge and the wearers face.
4. Leave the working area immediately if breathing becomes difficult or dizziness or any other signs of distress occur.
5. It is not allowed to use the respirator without proper fit.
6. The instruction for use and warning is helpful for correct using of the product and to prevent risks of health. It is preemptry to read it carefully and to follow it.

INSTRUCTIONS FOR FITTING

1. **Prepare to use respirator**
 - Check the Respirator and Hood for holes, tears or other damage to surfaces and straps to ensure that your Respirator works properly. Replace with a new set if damaged.
 - Take the Protective Hood <band> one after another, putting on the Hood over the head with the Respirator in the front.
2. **Place respirator on face**
 - Position the Respirator properly to cover your nose and mouth.
3. **Place the elastic band**
 - Place the top elastic band at the top back portion of your head. Next, place the bottom elastic around the neck below your ears.
4. **Adjust the elastic band**
 - If need be, pull the bottom elastic strap backwards and adjust the headband to a fitting and comfortable tension.
5. **Adjust the tension**
 - Hold the end of bottom elastic strap with both hands and pull at either side of buckle for better fit. To loosen elastic strap, pull buckle outwards.
6. **Adjust the velcro strap**
 - Pull velcro strap gently across front.
 - Adjust hood to allow strap to attach on corresponding side of fabric portion.
 - If you wear collared shirt, it is recommended to wear it over neck seal (see fig 7).
8. **Check the face seal**
 - Completely cover with both hands on the surface of the Respirator, then exhale or breathe out sharply.
 - If you feel air blowing on your face or eyes, the Respirator needs to be adjusted. To adjust, stretch the headbands again by pulling them backwards. Correct the position of the Respirator and check the face seal again. When the Respirator is a good fit, you will not feel any air blowing on your face or eyes.

INSTRUCTIONS FOR DISPOSAL

1. 
 - Remove the Hood by gently pulling upwards with your fingers.
2. 
 - Hold the inner top of the Hood with one hand, with the other hand holding the bottom strap and pulling it downwards. Avoid touching surface of Respirator.
3. 
 - Pull the Hood inside out. This is to reduce exposure of contaminants.
4. 
 - Fold the Hood in layers for proper disposal.
5. 
 - Dispose off spent Protective Hood in line with proper hygienic guidelines.

WARNING: IT IS IMPORTANT TO USE THE RESPIRATOR WITH PROPER FIT.

Champak Enterprise Co., Ltd
No 27-1, Jhaiming St, Dasi Town, Taoyuan County,
335, Taiwan
Web: www.champak.com.tw
Item No: A520H

🌡️ -5°C ~ +50°C 🌧️ <60% 📖 LOT :