


Dräger X-plore 1300 Particulate Respirators

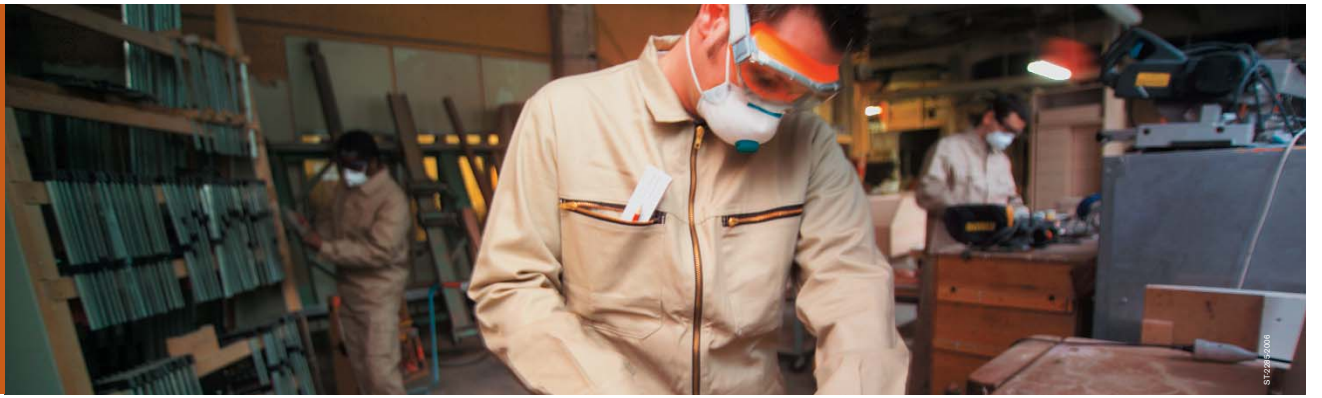

1 Hold the mask in your hand with the head straps hanging down.



2 Position the mask under your chin and over your nose and slip the short strap over your head to the back of your neck.



3 Pull the longer strap into position at the back of your head. Slide the adjustment knob toward your head for an optimal fit.



4 Adjust the nose clip to minimize leakage.



5 Cover the mask with both hands and exhale. The half-mask should inflate slightly. If any air escapes at the sides of the mask, readjust the mask until it fits properly and ensure that the nose clip is positioned correctly on your nose.



Ready for use.