

Arbill A501760

Arbill A501780

Gateway 80301, 80302V, 80304CV

Jinfuyu Kuranbon 1803

JSP 121, 122, 123

## **Donning Procedures**

1. Hold the mask in your hand with the nose piece at your fingertips and press it firmly against your face, with the nose piece on the bridge of your nose.
2. With your other hand, take the lower band, pull it over your head and position it below your ears.
3. Position the upper band in the same manner but place it above your ears.
4. Using both hands, mold the metal nosepiece to the shape of your nose and exhale vigorously. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, re-position the bands for better fit.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor for proper fitting respiratory protection.

JSP 821, 822, 823

## Donning Procedures

1. Hold the respirator in one hand with nosepiece at your fingertips, allowing the headbands to hang below your hand. The harness clip should be unsecured.
2. Press the respirator against your face with the nose part of the mask resting on the bridge of your nose.
3. Place the top band high at the back of your head.
4. Pull the two parts of the lower band back behind the neck and secure the clip. Adjust the tension of the strap by pulling the top band up from the head and sliding the tension adjuster down to the head to achieve a comfortable fit.
5. Test the fit of the respirator. Cup both hands over the respirator and exhale vigorously. If air leaks around the edges of the mask, tighten and/or reposition the bands for a better fit. Repeat this test.
6. Remember, careful observance of these fitting instructions is an important step for safe respirator use. If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor for the instruction to obtain the proper fitting of your respirator.

JSP 321, 322, 323

## Donning Procedures

1. Mould the nosepiece to the shape at your fingertips, allowing the headbands to hang below your hand.
2. Press the respirator against your face with the nosepiece on the bridge of your nose.
3. Place the top band high on the back of your head. Move the bottom band over your head and position it below your ears.
4. Using both bands, mould the nosepiece to the shape of your nose.
5. Test the fit. Cup both hands over the respirator and exhale vigorously. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the bands for better fit.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor for proper fitting respiratory protection.

## South West Safety

7150, 7151, 7152, 7153, 7154, 7156, 7157, 7160, 7177

# Donning Procedures

1. Hold the respirator in one hand with the nosepiece at your fingertips, allowing the headbands to hang below your hand.
2. Press the respirator against your face with the nosepiece on the bridge of your nose.
3. Place the top band high on the back of your head. Move the bottom band over your head and position it below your ears.
4. Using both hands, mould the nosepiece to the shape of your nose.
5. Test the fit. Cup both hands over the respirator and exhale vigorously. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the hands for better fit.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor for proper fitting respiratory protection.