


CAUTION

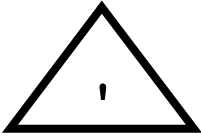

This respirator helps to protect against certain particulates. The improper use can cause illness or death.

“See NIOSH Approval label for further information.”

To occupational use of this respirator as part of a OSHA respirator program, each individual must pass either a qualitative or quantitative fit test.

Made in COLOMBIA BY



Calle 107 # 37-39

PBX: + 57 5 382 16 16

e-mail :

ingenieriatecnica@indusaver.com

NIOSH APPROVED



NIOSH APPROVED N-95

Ref. P/N LIFE 1095

INSTRUCTIONS

Fitting instructions to be followed each time the respirator is worn:

1. Hold the respirator in your hand, allowing the headbands to hang freely below your hands.



2. Position the respirator comfortably under your chin and against the bridge of your nose.



3. Pull the top headband over your head, placing it at the crown of your head. Pull the bottom headband over your head and place it below your ears.

4. USING BOTH HANDS, mold the metal nosepiece to the shape of your nose and cheek area. Using one hand to pinch the nosepiece may cause a bad fit, resulting in lower effective respirator performance.



5. FIT CHECK: Place both hands completely over the respirator. Inhale sharply. Be careful not to disturb the position of the respirator. A negative pressure (mask wanting to collapse) should be felt. If not, or if air leaks around the edges of the respirator, reposition the respirator and headbands for a better fit. If unable to adequately seal filter with hands, obtain the optional respirator fit check cup to perform this test.



If you CANNOT achieve a proper fit, **DO NOT WEAR THIS MASK**