# N95 Particulate Respirator

Minimum 95% Filtration Efficiency against Solids and Oil-free Liquid Aerosols

**INTENDED USE:**  
This mask is designed to provide 95% filter efficiency and respiratory protection for the wearer. The mask reduces but does not eliminate risk of exposure to airborne biological particles, infection or sickness.

**NOT FOR USE:**  
This mask is not intended for industrial or mining operations. For single use in surgical situations

Before use, must be fit tested either quantitatively or qualitatively by a qualified fit tester before using the respirator in a contaminated environment, as per the OSHA guidelines 1910.134

**LIGHT-WEIGHT, FLUID REPELLENT, LATEX-FREE**

**TIME USE RESTRICTIONS:**

1. Do not use the same respirator for more than 8 hours, continuous or intermittent, in dirty workplaces that could result in high filter loading.
2. Leave contaminated area and remove respirator if breathing becomes difficult or proper fit cannot be obtained or the respirator is soiled or damaged.
3. The maximum service time of this respirator is 5 consecutive calendar days (including days of non-use), beginning from the first day of use.

**FOR MULTIPLE USE ONLY IN ACCORDANCE WITH THE FACILITY'S INFECTION CONTROL POLICY**

The wearer must read Cautions & Limitations, Fitting Instructions, and be properly trained according to applicable Safety & Health Standards of the Facility before using the respirator

**FITTING INSTRUCTIONS**

1. Cup the respirator in hand with the nosepiece over the fingertips. The headbands must hang free below the hand
2. Place the respirator firmly against the face, positioning under the chin with the nosepiece upwards. Stretch, pull over and position the upper headband high over the back of the head. Next, stretch and pull the lower headband over the head positioning it below the ears.
3. Using the fingertips of both your hands on the two sides of the respirator, gently push the metal nosepiece against the shape of your nose. Move your fingers up and down the side of the nose for a snug and comfortable fit.
4. To test the fit, cup the respirator completely with both your hands without disturbing its position. Exhale energetically and if air leaks around the nose, push in and readjust the metal nosepiece. If air leaks from under the edges of the respirator, stretch and re-position both the headbands. The respirator is safe for use only if a proper fit is achieved.

**REMOVAL INSTRUCTIONS**

Without touching the surface of the respirator, carefully and slowly lift the bottom and then the top headbands. Store for re-use or discard as per the Facility's Infection Control Policy.

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\*PROTECTION: N95 - particulate filter (95% filter efficiency level) effective against particulate aerosols free of oil; Time Use Restrictions may apply.

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**2.CAUTIONS & LIMITATIONS:**  
A-Not for use in atmospheres containing less than 19.5% oxygen.  
B-Not for use in atmospheres immediately dangerous to life or health.  
C-Do not exceed maximum use concentrations established by regulatory standards.  
J-Failure to properly use and maintain this product could result in injury or death.  
M- All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.  
N- Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.  
O- Refer to Users'Instructions, and/or Maintenance Manuals for information on use and maintenance of these respirators.  
P- NIOSH does not evaluate respirators for use as surgical masks.  
S- Special or Critical Users'Instructions and /or Specific Use Limitations apply. Refer to Users'Instructions before donning.

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S- Special or Critical User's Instruction: The model FUT9500 filtering face piece respirator has been manufactured by Approval Holder Makrite for FUTUREMED HEALTHCARE PRODUCTS LP under TC-84A-3323. Manufactured for / Fabriqué pour : FUTUREMED HEALTHCARE PRODUCTS LP 277 BASALTC ROAD CONCORD ON L4K 5V3

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