

Dräger X-plore 1700 Particulate Respirators



DONNING THE MASK



1 Hold the mask in your hand with the head straps hanging down.



2 Position the mask under your chin and over your nose. Slip the first strap over your head to the back of your neck.




3 Pull the upper strap into position at the back of your head, ensure the strap sits above your ears.



4 Adjust the nose clip to minimize leakage.

USER SEAL CHECK

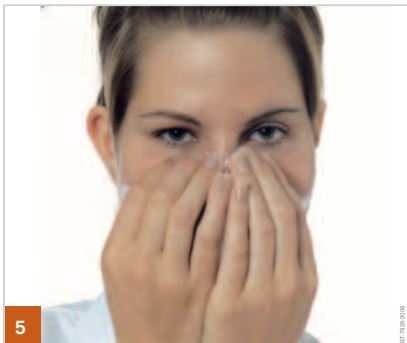

5 Cover the mask with both hands and exhale. The mask should inflate slightly. If any air escapes at the sides of the mask, readjust the mask until it fits properly and ensure that the nose clip is positioned correctly on your nose.



Ready for use.

Warning: Beards in the sealing area of the mask can cause leaks. Always make sure the mask forms a tight seal prior to entering a hazardous atmosphere.

This poster is not a replacement for reading and understanding the instructions for use prior to using the specified mask! Failure to do so could result in sickness or death! For further assistance concerning proper use, ask your supervisor or call your local Dräger organization at +1 800 922 5518.