FIT TEST INSTRUCTIONS 8145P95

1. Hold the respirator in your hand with the metal adjusting strip away from you, allowing the headbands to hang below your hand.
2. Press the respirator firmly against your face with the narrow end on the bridge of your nose.
3. Pull the top strap over your head and position it high on the back of your head. Pull the bottom strap over your head and position it below your ears.
4. Adjust the top and bottom strap lengths using both hands to pull the ends evenly towards the back of your head. Straps may be loosened by pushing the back of the buckle outward.
5. Using both hands, mold the metal strip to the shape of your nose.
6. Check the facepiece-to-face seal by cupping both hands over the respirator and inhale vigorously. A negative pressure should be detected inside the facepiece. If air leaks in from around your nose, reform the metal strip. If air leaks in from around the edges, reposition the straps. Repeat as necessary until no air leak is detected, and you have a satisfactory seal. If you cannot achieve a good face-to-facepiece seal do not enter the contaminated area and see your supervisor.

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INSTRUCTIONS / INSTRUCCIONES

1. Hold the respirator in your hand with the metal adjusting strip away from you, allowing the headbands to hang below your hand.
2. Press the respirator firmly against your face with the narrow end on the bridge of your nose.
3. Pull the top strap over your head and position it high on the back of your head. Pull the bottom strap over your head and position it below your ears.
4. Adjust the top and bottom strap lengths using both hands to pull the ends evenly towards the back of your head. Straps may be loosened by pushing the back of the buckle outward.
5. Using both hands, mold the metal strip to the shape of your nose.
6. Check the facepiece-to-face seal by cupping both hands over the respirator and inhale vigorously. A negative pressure should be detected inside the facepiece. If air leaks in from around your nose, reform the metal strip. If air leaks in from around the edges, reposition the straps. Repeat as necessary until no air leak is detected, and you have a satisfactory seal. If you cannot achieve a good face-to-face seal do not enter the contaminated area and see your supervisor.

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