### Fitting Instructions:

1. Hold the respirator by cupping it in the hand with the narrow end away from you and the headbands hanging freely outside the hand.
2. Hold the top headstrap in the other hand and bring the respirator to the face, placing the chin in the large, rounded end of the respirator. Tilt the respirator until the narrow end touches the bridge of the nose. Position the respirator until the respirator makes comfortable contact at all points of the face.
3. Press the respirator against the face, then pull the top headband over the head and position so it fits securely at the top of the crown of the head. Pull the bottom headband over the head and down past the ears until it rests on the neck.
4. Gently move the respirator on the face until the face seal touches comfortably on the face all along its length. If glasses are to be worn, the facepiece should be low enough on the face not to interfere with the spectacle frame.
5. Carefully adjust the metal nose clip to fit snugly on the bridge of the nose and cheek area.
6. Before entering a hazardous atmosphere, conduct OSHA compliant fit testing as part of your total respiratory program; use U.S. Safety Bitrex Fit Testing Kit, P/N 79260.

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