Fitting Instructions (to be followed each time the respirator is worn)

1. Open the respirator from its folded position with both hands. Form a generous radius in the nose clip using both thumbs and forefingers.
2. Position both head straps over front of the respirator. Place the respirator comfortably under your chin and against the bridge of your nose.
3. Place the top head strap at the crown of your head. Place the bottom strap below your ears.
4. Place the forefinger of one hand firmly on the nose clip at the bridge of nose. Using the thumb and forefinger of the other hand, squeeze the nose clip firmly against the sides of the nose.
5. The mask should fit snugly around the perimeter. If it does not, reposition the mask, adjust the straps, and reshape the nose clip.
6. FIT CHECK: Place both hands completely over the respirator. Inhale vigorously. Be careful not to disturb the position of the respirator on the face. A negative pressure (mask wanting to collapse) should be felt. If not, or if leakage is felt around the edges of the respirator, reposition the respirator and/or reposition the headbands and/or re-adjust the nosepiece for a better fit.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor for proper fitting respiratory protection.

Applications

Particulate aerosols free of oils meeting the requirements described herein.

Preparations for Use



1. A properly selected respirator is essential to protect your health. Before using this product, consult your supervisor and review the instructions. If uncertain about the suitability of this respirator for your intended use, call MSA Safety Works at 1-888-MSA-HOW2 (1-888-672-4692).
2. Before use, the wearer must be trained by the employer in proper respirator use.
3. OSHA (29 CFR 1910.134) requires that each wearer of this respirator have a qualitative or quantitative fit test performed to determine the amount of protection it will provide.

Respirator Use Limitations - Do not exceed the following:

1. MAXIMUM USE CONCENTRATION:
 - a) 10 times the OSHA permissible exposure limit (PEL), or other governmental limitation, for the contaminant(s) present.
 - b) Immediately dangerous to life or health (IDLH) concentration for any contaminant present.
2. 8 hours of use (continuous or intermittent).
3. Any applicable regulatory or industry standard limitation or any limitation contained in the above warning.

Replace the respirator when:

- Unable to pass the fit check
- Breathing resistance is too high
- Time use limitation is reached

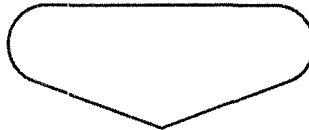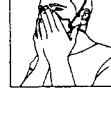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▲ WARNING

1. Failure to select the appropriate respirator for all contaminants and their concentrations, or failure to follow all warnings and instructions including the accompanying NIOSH Cautions and Limitations, may result in exposure to hazardous materials and risk serious injury, illness or death.
2. DO NOT use under any of the following conditions:
 - a) For protection against silica (sand), asbestos, lead, or other substances reported to cause cancer.
 - b) For protection against gases or vapors, paint spray, unknown contaminants, or any contaminant whose concentration is unknown or immediately dangerous to life or health (IDLH).
 - c) In areas not adequately ventilated or containing less than 19.5% oxygen.
 - d) While performing or if near sandblasting, concrete or masonry operations, building demolition, firefighting, or any operation that may produce airborne particles containing substances reported to cause cancer. Small particles are not visible and may remain airborne even after operation has ceased. DO NOT use where these airborne particles may be present.
 - e) In atmospheres where the concentrations of contaminants exceed the Maximum Use Concentration or if the governmental exposure limit (PEL, REL, TLV) is not known.
 - f) When conditions prevent a good face seal such as beards, large sideburns, deep scars, missing teeth, or other facial characteristics that prevent direct contact between the skin and the sealing surface of the respirator.
 - g) If not trained in respirator use.
3. Immediately leave contaminated area if:
 - Breathing becomes difficult
 - Dizziness or other distress occurs
 - You smell, taste, or sense irritation from the contaminant
 - The respirator becomes damaged
 - You develop an allergic reaction to the respirator material
4. This respirator will NOT eliminate exposure to contaminants. It is designed to reduce the breathing of contaminants by the wearer to an acceptable concentration when properly selected, fitted, and used. If a higher level of respiratory protection is desired, use an air-supplied respirator or full facepiece air-purifying respirator.
5. NEVER alter or modify this respirator.

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FAILURE TO FOLLOW THE ABOVE WARNINGS CAN RESULT IN SERIOUS PERSONAL INJURY OR DEATH.

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