NIOSH Approved N99 Particulate Respirator 2150 N99 Respirator

NIOSH
National Institute for Occupational Safety and Health

LOUIS M. GERSON CO., INC.
MIDDLEBORO, MA
1-800-225-8623

These respirators are approved only in the following configurations:

TC-	Protection ¹	Respirator	Cautions and Limitations ²
84A-XXXX	N99	2150 X	ABCJMNOP

1. Protection

N-99 Particulate filter (99% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.

2. Cautions and Limitations

A - Not for use in atmospheres containing less than 19.5% oxygen.
 B - Not for use in atmospheres immediately dangerous to life or health.
 C - Do not exceed maximum use concentrations established by regulatory standards.
 J - Failure to properly use and maintain this product could result in injury or death.
 M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
 N - Never substitute, modify, add or omit parts. Use exact replacement parts in the configuration as specified by the manufacturer.
 O - Refer to user instructions and/or maintenance manuals for information on use and maintenance of these respirators.
 P - NIOSH does not evaluate respirators for use as surgical masks.

IMPORTANT NOTICE TO PURCHASER

Manufacturer warrants this product to be free from defects in material and workmanship. In the event that this product proves, upon inspection, to contain defects in material or workmanship, Manufacturer undertakes, at its option, to replace any portion proving defective or to refund to the Buyer the purchase price paid by Buyer for this product. This warranty and Buyer's remedy of replacement or refund constitute the Manufacturer's exclusive warranty and the Buyer's sole remedy, in lieu of all other warranties expressed or implied. Except as provided above, Manufacturer shall not be liable for any loss or damage, whether direct, consequential, or incidental occasioned by the sale, use or misuse of this product, other than death or personal injury directly resulting from Manufacturer's negligence. Buyer shall determine the suitability of this product for its intended use and except provided above, assumes all risks and liability in connection therewith.

LOUIS M. GERSON CO., INC.
Middleboro, MA 02346 U.S.A.
© 2009 All Rights Reserved
Made in U.S.A. Pat. Pend.

551263 rev A

www.GersonCo.com

WARNINGS

Read and understand before use.

- A properly selected respirator is essential to protect your health. Before using this or any product consult an Industrial Hygienist or Occupational Safety Professional to determine its suitability for your intended use.
- This product will not protect against gases or vapors.
- Do not use in spray paint operations.
- Do not use in or around sand blasting operations.
- Do not use for protection against asbestos.
- This product does not supply oxygen. Use this respirator in a well ventilated area containing sufficient oxygen to support life.
- Do not use when concentrations of contaminants are unknown or immediately dangerous to life.
- Leave work area immediately if:
 - breathing becomes difficult, or
 - dizziness or other signs of distress occur.
- If user has any illness or disorders, consult a licensed medical doctor to determine suitability of use.
- Facial hair and certain facial characteristics may prohibit effective use of this product.
- Use only in accordance with instructions, labels and limitations relating to this product.
- Do not alter or modify this product in any way.
- Follow the manufacturer's instructions for fitting the respirator to the face.

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Meets CDC Guidelines for TB & Influenza Control

Cat. No. 082150C

2150 N99 Respirator

NIOSH Approved N99 Particulate Respirator

Meets CDC Guidelines for TB & Influenza Control

- Folds Flat For Easy Storage and Portability
- Latex Free
- Durable, Ultrasonically Welded Head Bands
- Fluid Resistant

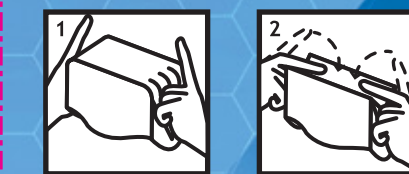
Respirator Use Instructions: Instructions d'utilisation du Respirateur: Instrucciones para el uso de Respiradores:

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Folding Instructions: Instructions du Pliage: Instrucciones para Doblar:

NOTE: Before issuing any respirator to be worn, a face fit test must be performed. See OSHA regulations 1910.134.

Glue ----- K/O

K/O

K/O

2.4in
60.96mm

11.375in
288.925mm

Dieline

2.35in
59.69mm

5.75in, 146.05mm

3.625in, 92.075mm

5.75in, 146.05mm

3.625in, 92.075mm