

Donning Procedures

1. Mould the nosepiece to shape the respirator with your fingertips, allowing the headbands to hang below your hand.
 2. Press the respirator against your face with the nosepiece on the bridge of your nose.
 3. Place the top band high on the back of your head.
 4. Using both hands, mould the nosepiece to the shape of your nose.
 5. Pull up or down the band through ring adjuster to tune the tightness of band.
 6. Test the fit.
Cup both hands over the respirator and exhale vigorously.
If air flows around your nose, tighten the nosepiece.
If air leaks around the edges, reposition the bands for better fit.
- a) More compact folding can be achieved by pushing in arrowed direction and pleating along with emboss lines.
- b) The plastic band hooks can be used to tie on or tie off the band.

See your supervisor for proper fitting respiratory protection.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area.

See your supervisor for proper fitting respiratory protection.

