

Donning Procedures

1. Before every single use, replace a new filter onto the facepiece.
2. Slip the side flap of the facepiece into the matching slot on the filter. Do it again for the other side. Make sure the filter has been sit properly onto the facepiece completely.
3. Hold the respirator in one hand and pull the headbands by the other hand. Press the respirator against your face with the nose foam on the bridge of your nose, pulling the headbands over your head to locate.
4. Position the top band high on the back of your head. Move the bottom band below your ears.
5. Adjust the strap tension by pulling the straps as shown in the figure.
6. Test the fit. Cup both hands over the respirator and exhale vigorously. If any leakage is detected, readjust the position of the respirator according to steps #4, #5 and #6 till get a better fit.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor for proper fitting respiratory protection.

