

Arbill A501760

Arbill A501780

Gateway 80301, 80302V, 80304CV

Jinfuyu Kuranbon 1803

JSP 121, 122, 123

Donning Procedures

1. Hold the respirator in one hand with the nosepiece at your fingertips, allowing the headbands to hang below your hand.
2. Press the respirator against your face with the nosepiece on the bridge of your nose.
3. Place the top band high on the back of your head. Move the bottom band over your head and position it below your ears.
4. Using both hands, mould the nosepiece to the shape of your nose.
5. Test the fit. Cup both hands over the respirator and exhale vigorously. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the bands for better fit.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area.
See your supervisor for proper fitting respiratory protection.

