Ref.: ZZM002US
42 CFR 84
EN149:2001
(C0070)

Developed by French National Institute for Occupation of Safety and Health

Instructions for use of respiratory masks N95 for single use

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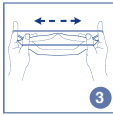
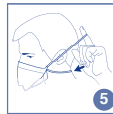
Description and indications:

MacoPharma Respiratory Protection Masks N95 for single use meet the requirements of the 42 CFR 84 standard.
These filtering masks are recommended to any person (health care professionals, patients, visitors...) looking for a protection against particulate aerosols free from oil.

Instructions for use:

This mask may only be used by people who were trained to use it. Take the mask out of its pack at the time of using it only. Before using it, ensure that it is intact and that elasticised strings are intact too.

To ensure that the respiratory mask tightly adheres to your face:

- Cup your hands over the mask to block up its surface of filtration as much as possible, without moving it.
 - Breathe out really strongly.
 - if any air flows around your nose, readjust the nose bar as in picture 7
 - if any air flows out around the mask, carefully adjust the strings to the back.
- In case tightness obtained is not satisfactory, do not enter the area where it is required to wear the mask.

Warning:

- It is unlikely that tightness requirements are met if hairs from your beard stick out under the face seal.
- Leave the premises if:
 - you feel dizzy or feel any pain;
 - it becomes hard to breathe.

Dispose of and replace the mask if:

- it is damaged;
 - it seems to hamper your breathing;
 - it is tainted by blood or any other infectious substance.
- Dispose of the mask via the Infectious Hazard Care Activity Waste circuit.

Use restrictions:

This mask does not provide any protection against gases or fumes. Do not use this mask if the atmospheric pollutant concentration exceeds 9 times its Exposure Limit Value. This half mask does not supply any oxygen. Do not use in atmospheres which do not contain at least 19.5% oxygen. Do not use in an explosive atmosphere.

This mask does not require any maintenance. It has been designed for one single continuous work session, i.e. for 8 hours as a maximum.

Each user must be fit tested either qualitatively or quantitatively as per the OSHA 1910.134 guidelines.

Any failure to comply with instructions and warnings concerning the use of this respiratory protection mask for the whole period of exposure may reduce its efficiency. This product does not eliminate the risk of contracting diseases or infections.

Conformités:

This mask meets requirements defined by European Directive 89/686/EEC. It has been examined by the Institut National de Recherche et de Sécurité, Avenue de Bourgogne 54 500 Vandoeuvre, France (identification number: 0070). This mask meets extra requirements as stipulated by the Journal Officiel de la République Française, 28 September 2005.

This respirator does not contain any latex.

Storage:

Store according to recommendations as displayed on the pack.

Best before date: 2 years

Single use

Temperature range for storage conditions: 0°C to 40°C

Maximum moisture for storage conditions: 85%

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THIS RESPIRATOR IS APPROVED ONLY IN THE FOLLOWING CONFIGURATION:

TC	Protection	Respirator	Cautions and Limitations ¹
84A-MAC1192007	N95	X	ABCJMNP0
ZZM002US			

1. Protection

N95 - Particulate Filter (95% filter efficiency level)
Effective against particulate aerosols free of oil;
time use restrictions may apply

2. Cautions and Limitations

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N - Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to users instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.