


FITTING INSTRUCTIONS

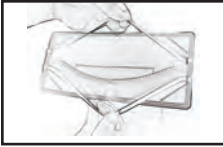

Step 1

Move both headbands to the outside of the respirator.




Step 2

Unfold the respirator (as illustrated) by pulling the upper part upward and the lower part downward.



Step 3

Put the respirator lightly against your face with its nose clip of the upper part fit across your nose and the lower part fit under your chin.

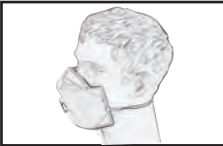

Step 4

Pull the lower headband over your head and fix it around the upper part of your neck.




Step 5

Pull the upper headband and fix it on around the upper part of your head. Be careful lest both bands should cross each other. The yellow band must be above the white one.



Step 6

Adjust the position of the respirator so that it is located in the center of your face and press the nose clip along the curves of your nose and cheeks. Then, breathe and ensure no leakage of air. Especially around your nose and chin, and in front of your ears are places where air tends to leak easily.

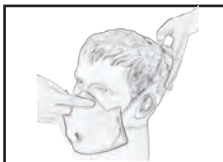

Step 7

If you find air leakage between the respirator and your face, press the nose clip again in order to fit the respirator perfectly or adjust the positions of the headbands until no leakage is confirmed.