

55-0935S Rev 4  
11/16/2009

**NIOSH Approved**  
**N95 Particulate Respirator**  
**65-3395S**  
**Precept**

**Precept** 65-3395S

**Warnings**

- This product does not eliminate the risk of contracting any disease or infection.
- A properly selected respirator is essential to protect your health. Before using, the wearer must be trained by the employer in accordance with safety and health standards.
- Leave work area immediately if: (A) breathing becomes difficult, or (B) respirator becomes damaged.
- Facial hair and certain facial characteristics may prohibit effective use of this product.
- Use only in accordance with instructions, labels and limitations relating to this product.
- Do not alter or modify this product in any way.
- Follow the manufacturer's instructions for fitting the respirator to the face before each use.
- Filtering facepieces are to be inspected prior to each use to insure that there are no holes in the breathing zone and no damage has occurred.
- "S" SPECIAL INSTRUCTIONS  
The model 65-3395S filtering facepiece respirator has been manufactured by Makrite for Precept Medical Products Inc. under TC-84A-4107.

**Respirator Fitting Instructions**

OSHA Standard 29 CFR 1910.134 (e)(5) requires that the wearer be fit tested.  
The Saccharin Qualitative Fit Test is recommended for this respirator.



1. Hold the respirator in hand with the nosepiece at your fingertips, allowing the headbands to hang freely below your hand. Press the respirator firmly against your face with the nosepiece on the bridge of your nose.



2. Stretch and position the top band high on the back of your head. Stretch the bottom band over the head and position below your ears. Using both hands, mold the metal nosepiece to the shape of your nose.



3. To test fit, cup both hands over the respirator and exhale vigorously. If air leaks, tighten nosepiece or reposition straps for a better fit. Remember, careful observation of these fitting instructions is an important step in safe respirator use.

Latex Free



Precept Medical Products, Inc.  
1-800-438-5827 www.preceptmed.com

**Precept** 65-3395S

**NIOSH Approved**  
**N95 Particulate Respirator**



Precept Medical Products  
370 Airport Road, Arden, NC 28704  
800-438-5827



This respirator is only approved in the following configuration:

TC	Protection <sup>1</sup>	Respirator	Cautions and Limitations <sup>2</sup>
84A-4107	N95	65-3395S	x ABCJMNOPS

1. Protection

N95 - Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil. Time use restrictions may apply.

2. Cautions and Limitations

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N - Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to users instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- P - NIOSH does not evaluate respirators for use as surgical masks.
- S - Special or critical User's Instructions and/or specific use limitations apply. Refer to User's Instructions before donning.


Latex Free

LOT

55-0935S Rev 4

Precept Medical Products, Inc.  
1-800-438-5827 www.preceptmed.com

**Precept** 65-3395S

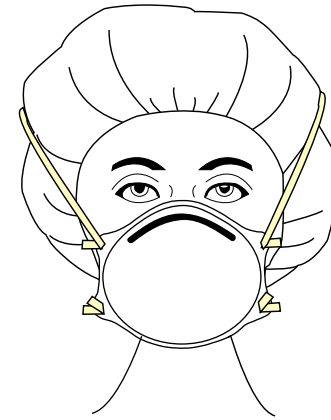

**N95 Particulate Respirator (Small)**

Color: White  
Packaging: 20/Box

Latex Free

Passes ASTM F 1862 at 160 mm Hg  
Meets CDC Guidelines for TB Exposure Control

Precept Medical Products, Inc.  
1-800-438-5827 www.preceptmed.com

**Precept** 65-3395S

**NIOSH Approved**  
**N95 Particulate Respirator**

The Precept 65-3395S Type N95 NIOSH Approved Particulate Respirator is a no fiberglass, fluid resistant, single-use mask that provides respiratory protection for the wearer. This respirator was tested and approved by NIOSH to have a filter efficiency level of 95% or greater against solid particulates.

**Important Notice to Purchaser**

The following warranty is made in lieu of all other warranties expressed or implied, including but not limited to, the implied warranties of merchantability and fitness for purpose. Manufacturer warrants this product to be free from defects. Manufacturer's only obligation shall be, at our option, to replace any portion proving defective or to refund the purchase price thereof. USER SHALL DETERMINE THE SUITABILITY OF THE PRODUCT FOR HIS INTENDED USE, AND ASSUME ALL RISKS AND LIABILITY IN CONNECTION THEREWITH.

Latex Free

Meets CDC Guidelines for TB Exposure Control

Precept Medical Products, Inc.  
370 Airport Road Arden, NC 28704 1-800-438-5827 Made in China