FITTING

1. Users must follow instructions each time respirator is worn.
2. It's the wearer's responsibility to obtain a proper fit.
3. To choose the correct model/size Moldex Respirator you must fit test and user seal check.
4. If you cannot obtain a proper fit do not enter the contaminated area.
5. OSHA regulation 29 CFR 1910.134 (f) requires that the user must be fit tested before being assigned a respirator and annually thereafter.
6. Use Moldex Qualitative Fit Test Kit. User must be clean-shaven. Any facial hair, such as beards or long sideburns, may prevent the respirator from fitting properly.
7. Eyewear must not interfere with face to facepiece seal.
8. Re-fit test when there are any changes to the wearer's physical condition that may affect fit.

1

1. Hold respirator in hand with molded nose contour (narrow end) at finger tips, allowing headstraps to fall below hand.
2. Place respirator under chin with molded nose contour (narrow end) up. Nose cushion must be increased inside respirator. Raise top strap to top back of head. Pull shorter bottom strap over head, below ears, to around neck. **Do not wear with only one strap. When changing from any model/size to another respirator, you must fit test.**
3. Adjust respirator for comfortable fit.
4. Each time user enters contaminated area seal should be seal checked. Cover front of respirator by cupping both hands. **INHALE SHARPLY.** A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, adjust straps by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.

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