INSTRUCTIONS / INSTRUCCIONES

To Put On This Respirator: Mise en place du respirateur: Para ponerse este respirador:

Before using this respirator, the wearer should be trained in its proper use and have it fit tested to make sure it fits correctly. Instructions for training and fit testing are given in OSHA 29 CFR 1910.134. In Canada, the training program should be based on CSA Standard Z94.4-02.

1. Open the respirator by molding the top nose strip to the shape of your nose.
2. With your chin in the bottom of the respirator, pull the bottom strap over your head and below your ears.
3. Place the top of the respirator over your nose and pull the top strap over your head. Position the strap high on the back of your head.
4. Using both hands, mold the metal strip to the shape of your nose and position the respirator firmly against your face.
5. Check the facepiece-to-face seal by cupping both hands over the respirator and inhale vigorously. A negative pressure should be detected inside the facepiece. If air leaks in from around the edges, reposition the straps. Repeat as necessary until no air leak is detected, and you have a satisfactory seal. If you cannot achieve a good face-to-facepiece seal do not enter the contaminated area and see your supervisor.

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Time Use Limitations:

The use and reuse of this respirator is subject to considerations of hygiene, damage and increased breathing resistance. Replace if the respirator becomes damaged or if breathing becomes difficult.

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