INSTRUCTIONS / INSTRUCCIONES

1. Hold the respirator in your hand with the metal adjusting strip away from you, allowing the headbands to hang below your hand.
2. Press the respirator firmly against your face with the narrow end on the bridge of your nose.
3. Pull the bottom strap over your head and position it below your ears. Pull the top strap over your head and position it high on the back of your head.
4. Adjust the top and bottom strap lengths using both hands to pull the ends evenly towards the back of your head. Straps may be loosened by pushing the back of the buckle outward.
5. Using both hands, mold the metal strip to the shape of your nose.

6. Check the facepiece-to-face seal by cupping both hands over the respirator and inhale vigorously. A negative pressure should be detected inside the facepiece. If air leaks in from around your nose, reform the metal strip. If air leaks in from around the edges, reposition the straps. Repeat as necessary until no air leak is detected, and you have a satisfactory seal. If you cannot achieve a good face-to-facepiece seal do not enter the contaminated area and see your supervisor.

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Time Use Limitations:

The use and reuse of this respirator is subject to considerations of hygiene, damage, increased breathing resistance and gas or vapor breakthrough. Replace if the respirator becomes damaged, if breathing becomes difficult or you smell, taste or sense irritation from the contaminants in the work area. Service time should not be extended beyond 24 total hours of use.

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37.5" Please correct box size if needed

11.5"

NORTH Safety Products
www.northsafety.com

USA: 1-800-424-4110 CANADA: 1-800-321-5711 EUROPE & MIDDLE EAST: +31 (0) 118 95490
 1-818-883-4800 818-883-4800
 CHINA: 06-31-9792588-881 ASIA PACIFIC: 812-82252611 SOUTH AMERICA: 5711-4712043 AFRICA: 2711-4712043
 06-31-9792588-881 812-82252611 5711-4712043 2711-4712043
 Made in / Fabriqué en / Hecho en / CHINA

8145P95

UNIVERSAL • UNIVERSEL • UNIVERSAL
DISPOSABLE • JETABLE • DESECHABLE
UPC# 00821812000350