

USER'S INSTRUCTIONS FOR MODEL :RD5-V, PART # 9226-002

NOVEKO Inc.

This product does not contain any component made from natural rubber latex.

INTENDED USE:

- Design to help reduce wearer exposure to certain airborne particles.

PROTECTION: N95 – Particulate Filter (95% filter efficiency level), effective against particulate aerosol free of oil; time use restriction may apply.

USE LIMITATIONS & WARNINGS:

- Not for use in aerosol oil environment.
- Do not use the respirators if you have facial hair or any other conditions that may prevent a good seal on the face.
- If respirator leaks or does not fit properly, do not enter the contaminated area.
- Do not modify and/or misuse the respirators, it may result in sickness or death.
- Consult the side panel section "Cautions and Limitations" on the box for complete information.
- OSHA has not set a permissible exposure level for airborne biohazards.

SPECIAL CAUTION

- The respirator's shelf life is located at the bottom of the box.

USE INSTRUCTIONS:

- Follow the "Fitting Instructions" while making sure to check the fit by cupping the respirator with both hands and breathing heavily.
- Before occupational use, the wearer must be trained by the employer and a written respiratory protection program must be implemented in accordance with all the requirements of OSHA 29 CFR 1910.134 such as medical evaluation, training and fit testing.
- Immediately change the respirator and replace with a new one if:
 - Damaged (cut, torn, etc.)
 - Breathing becomes difficult
 - Contaminated with blood or body fluids
- Discard the respirator in accordance with local and Federal environmental requirements and replace with a new one after each use.

FITTING INSTRUCTIONS

Follow these instructions each time respirator is worn:



1.

Open the respirator and bend the nosepiece to form a slight curve.



2.

Holding the straps with both hands, place the respirator over your nose and mouth, with the top half over the bridge of your nose and the bottom half under your chin.



3.

Pull the straps over the top of your head. Rest the top strap on the crown of your head above the ears and fit the bottom strap below the ears.



4.

Mould the top half of the respirator to your face by pressing gently with both hands.



5.

Check the fit by cupping the respirator with both hands and breathing heavily. If respirator leaks or does not fit properly, do not enter contaminated area.