




Safe Life™ Disposable Respirators

Donning and Fitting Procedures

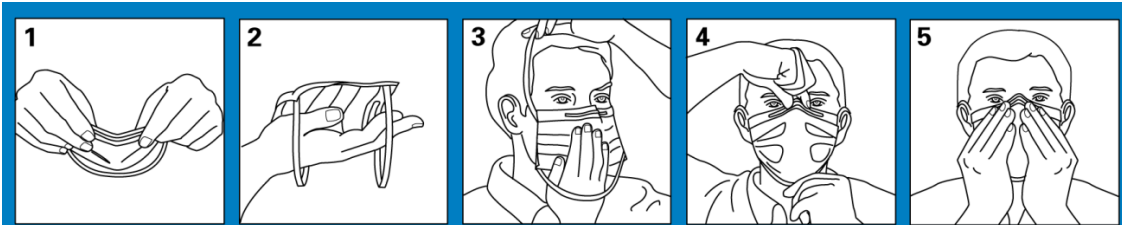

Donning and Fitting

- 1 Pre-shape the nose clip at the top center of the respirator as it will rest over the bridge of the nose.
- 2 Hold the respirator in one hand with straps dangling beneath and the inside of the respirator facing upward.
- 3 Hold the respirator against the face, pull the top head-strap over the head and position it above the ears. Pull the lower head-strap over the head and position it around the neck below the ears.
- 4 Hold the nose clip while pulling down on the bottom of the respirator to open the pleats and position the bottom of the respirator just beneath the chin. Press nose clip to form along the nose on both sides until a good seal is established.
- 5 **FIT CHECK:** Place both hands on the edges of the respirator, being careful not to disturb the position of the respirator. Exhale sharply to detect any leakage. If you detect air leaks, re-position or re-don the mask as needed to achieve a satisfactory seal.

Removal of Respirator

After use, microorganisms may be present on the exterior surface of the respirator. Remove the used protective device carefully, avoiding self-contamination, and immediately sanitize hands. Disinfect surfaces in the immediate area where droplets of microorganisms may have fallen.

Important

If you cannot achieve a satisfactory fit, do not enter the contaminated area. Failure to follow all instructions and limitations can seriously reduce the effectiveness of any respirator and could lead to illness, injury or disease. Respirators cannot completely eliminate the risk of contracting disease or infection.