


USER'S INSTRUCTIONS

For Model No. SH3600

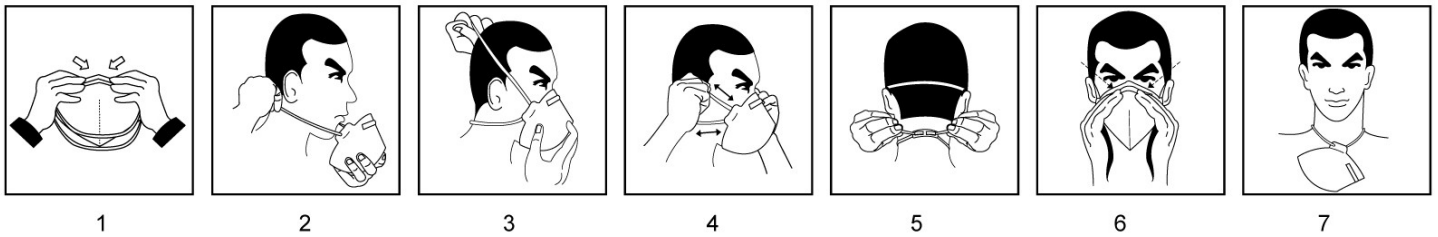

FITTING INSTRUCTIONS :

1. Place fingers inside the respirator. Bend the nose clip around fingers to form nose shape.
2. Hold mask in position over nose and mouth, with retaining straps around back of head. Pull lower portion of strap over head and fit around neck.
3. Stretch and pull the upper strap over your head on the back of your head above your ears.
4. Using both hands, form nose clip around nose and adjust the tension in the upper and lower portions of the continuous loop strap, to ensure a positive face seal.
5. Tension may be increased by pulling strap at either side of buckle to obtain a tighter facial fit. Tension may be decreased by pushing out the back of the buckle to obtain a looser facial fit.
6. Perform a User Seal Check prior to each wearing : Cup both hands over the respirator and exhale sharply. If air flows around your nose, tighten the nose clip. If air leaks around the edges, repeat point 4 and 5. **If you CANNOT achieve a proper face seal, DO NOT enter the contaminated area. See your supervisor.**
7. During work break, take off the lower portion of strap. Let the respirator hang around the neck.
8. Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Careful observance of these instructions is an important step in safe respirator use.

WARNING :

1. This product does not eliminate the risk of contracting any disease or infection.
2. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death.
3. Before occupational use of this respirator, a written respiratory protection program shall be implemented meeting all the requirements of OSHA 29 CFR 1910.134 such as training, fit testing, medical evaluation, and the applicable OSHA's substance specific standards.
4. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19.5%.
5. Do not use when concentrations of contaminants are immediately dangerous to health or life.
6. Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs.
7. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator.
8. Never alter or modify this respirator in any way.
9. Only for single use. No maintenance necessary. Discard the used respirator after single use.
10. Keep respirators in the display box away from direct sunlight until use.

Rev. A, 2008/06/03

Note : The NIOSH approval label and user instruction shall be supplied with the smallest commercial packaging. If the label or user instruction is not printed on the outside of the container, it will be included as an insert inside the container and the container shall indicate "NIOSH Approved – See Insert".