

SHIGEMATSU WORKS CO., LTD.

Donning Instructions for Model DD01-N95 Filtering Facepiece

IMPORTANT

Users must read the instructions and use the respirator following these instructions.

FITTING

● Check before donning

Users must check the following items before use.

The user is required to be fit tested according to OSHA 1910.134 either qualitatively or quantitatively prior to use.



If you find the nonconformity, please exchange for new one.

1. if there is damage.
2. if mask is stained or dirty.
3. if the straps do not hold the mask securely to the face.


● Donning Procedures

1. The mask is flatly folded by the rectangle.


Loosen head strap and unfolded the mask.

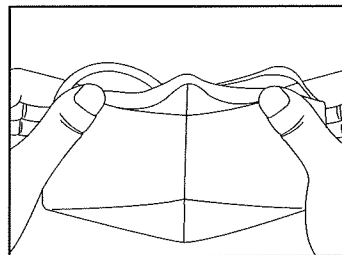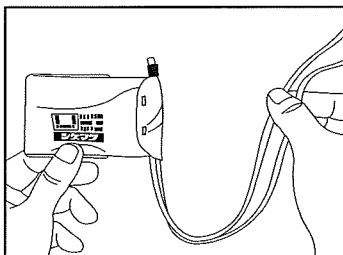

2. Extend the lower head strap and hang the mask to the neck.

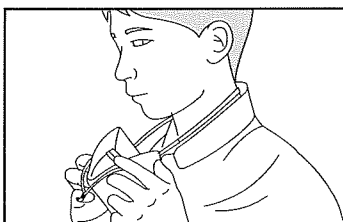



3. Turn the upper head strap to outside of the mask. And hold the mask in both ends of the mask to cover the nose and mouth.

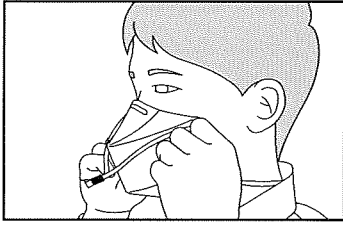


4. Hold respirator in one hand to cover the nose and hook the upper head strap to top of the head by the other hand.




5. Slide the adjuster in the upper head strap toward your head for an optimum fit to your face.

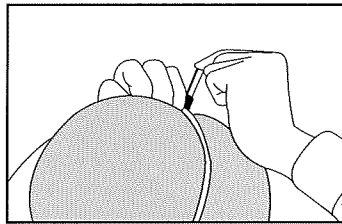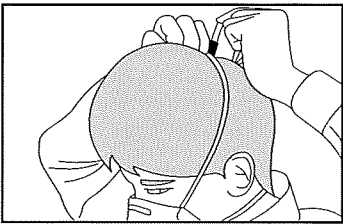

6. Adjust the tension of head straps confirming the fitting to your face.

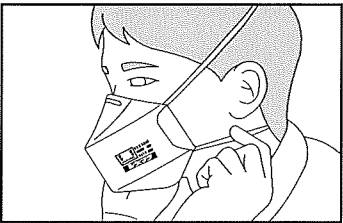

7. Adjust the nosepiece by your fingers of both hands and fit the mask to your nose.

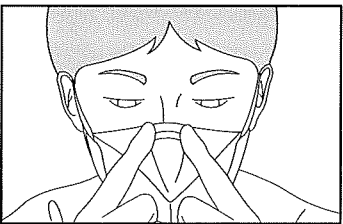

●SEAL CHECK PROCEDURES

Users are requested to select respirators which fit the face of the wearer in order to make sure the performance of the respirator.

Conduct the fit testing by the following procedures and please confirm the good fit before using.

1. Wear respirator same as in the work.
2. Exhale covering the respirator surface by both hands.
3. Please confirm no leakage of the exhalation breathing from facepiece when exhaling.

If wearer feels leakage, please adjust the fitting conditions until feeling no leakage.



APPLICATIONS

⚠ WARNING TO USER

1. Follow all instructions and warning on the use of this respirator.
2. The instructions are requested to be preserved where they can be reviewed when necessary.

USE AGAINST

1. This respirator is the disposable respirator to protect users from particulate aerosols raised in workplaces.

DO NOT USE AGAINST

1. Toxic gas and oxygen-deficient air (less than 19.5 %).

RESTRICTIONS

1. This respirator must not be used in atmosphere containing less than 19.5 % oxygen or toxic gas.
2. Concentrations of contaminants which are unknown, or are immediately dangerous to life or health.
3. Use respirator before the " use by " expiration date.
4. Use in dry condition.
5. Do not put anything inside of respirator.
6. Do not wash, if washed, efficiency will reduce.
7. Replace respirator without fail if it is stained, damaged or worn-out.
8. When the user inspects the welds in the breathing zone for cracks or holes. If there are holes or cracks the mask is damaged.
9. Dispose if respirator reaches the end of its service life, or if breathing resistance has noticeable increased.
10. The respirator is preserved in the place where it is clean, cool, dark and dry condition.

SHIGEMATSU WORKS CO., LTD.

3-13-8 Sotokanda, Chiyoda-ku, Tokyo
101-0021, JAPAN
TEL : +81-3-3255-0255
http : //www.sts-japan.com
E-mail : info@sts-japan.com