


Shuenn Bao Shing Corporation

AP0708V Disposable Particular Respirator

Fitting Instruction:

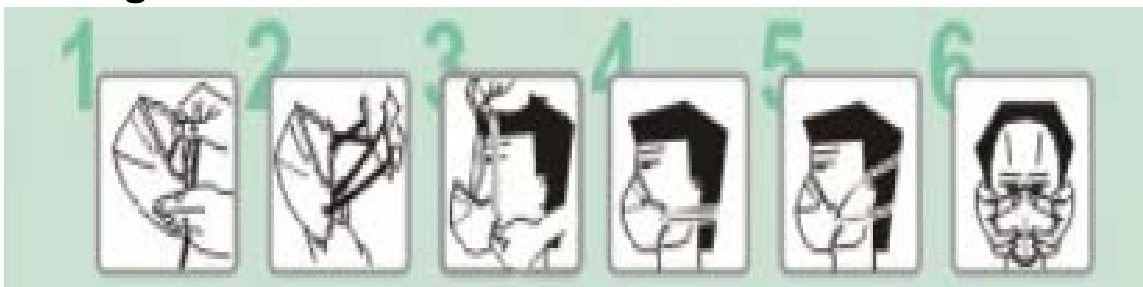

1 Open the mask .	2 Pull both elastic straps out from the mask.	3 Pass the elastics over the head and position the mask under the chin with noseclip facing upwards.	4 Position the lower elastic strap around the neck and release it	5 Position the upper elastic strap on the top of the head and release it	6 Adjust the mask over the face ensuring that pressing the nasal bar firmly and evenly against the nose for a snug and comfortable fit, so there are no leaks
------------------------------------	--	---	--	---	--

Test the tightness of the respirator by exhaling or blowing out sharply. Check for leaks around the nose clip and the rest of the seal of the respirator. Any leaks around the seal must be eliminated before entering the contaminated area.

It is the user's responsibility to ensure that the respirator provides the necessary level of protection for the type and concentration of the contaminants in the area where the respirator is intended for use.