

TOYO SAFETY 1795 (N95)

Donning Procedures

1. Put the respirator against your face with logo oriented your chin. Make sure the respirator is oriented correctly. Press the respirator against your face with the nose foam on the bridge of your nose.
2. Position the top band high on the back of your head. Move the bottom band below your ears.
3. Press the respirator against your face to make your face totally fit with the respirator.
4. Test the fit. Cup both hands over the respirator and exhale vigorously. If air leaks are detected, adjust the position of the respirator and check if air leaks again.

If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor for proper fitting respiratory protection.

