


ViralClyns™ *P99* **Respirator** with Antimicrobial Protection

Donning and Fitting Procedures

T-6000V™ (begin w/step 2)

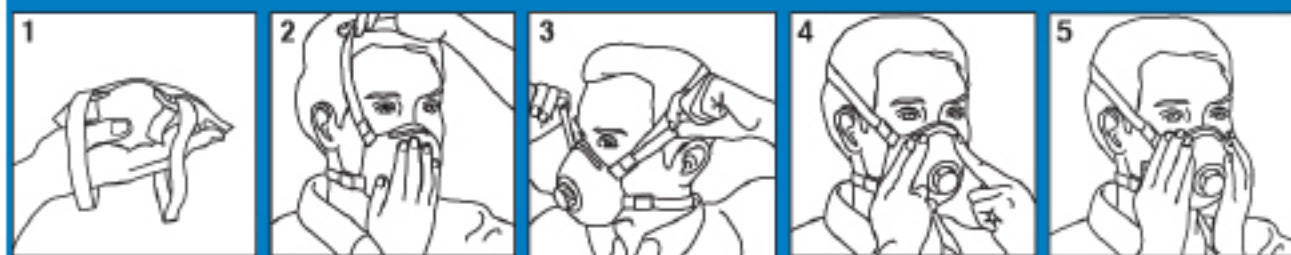

T-6000™ (begin w/step 2)



Donning and Fitting

- 1 Open the respirator and place forefinger under nosepiece. Mold nosepiece around finger.
- 2 Hold the respirator in the hand, allowing the headstraps to hang loosely.
- 3 Position the lower headstrap around the neck below the ears. Hold the respirator against the face and place the top headstrap around the crown of the head.
- 4 Adjust the headstraps by grasping the headstrap and pulling the adjusters until the respirator fits comfortably.
- 5 Fit the nose area to the shape of the face by forming the nosepiece with the fingertips. Using both hands, start from the top of the nosepiece, working downward while pressing against the nose.
- 6 **FIT CHECK:** Place both hands on the edges of the respirator, being careful not to disturb the position of the respirator. Exhale sharply to detect leakage. If you detect air leaks, readjust respirator and re-check the fit.

Removal of Respirator

After use, microorganisms may be present on the exterior surface of any mask or respirator. Remove the used protective device carefully, avoiding self-contamination, and immediately sanitize hands. Disinfect surfaces in the immediate area where droplets of microorganisms may have fallen.

Important

If you cannot achieve satisfactory fit, do not enter the contaminated area. Failure to follow all instructions and limitations can seriously reduce the effectiveness of any respirator and could lead to illness, injury or disease. Respirators cannot completely eliminate the risk of contracting disease or infection.

For more information, call 1-877-462-5967 or visit us at www.clynsbrands.com