



Jiangyin Changhua Industrial Manufacturing Co.,Ltd.

Address: North Industrial Park, Huangtang, Xiake Town Jiangyin City, Jiangsu,China.
Tel: +86-510-86539881 Fax: +86-510-86913736
E-Mail:marketing@jychanghua.com Web:http://www.jychanghua.com

This Respirator is approved only in the following configuration:

TC	Protection 1	Respirator	Cautions and Limitations 2
		NOJIMA-09	
84A-5230	N95	X	ABCJMNOP

1. Protection

N95-Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.

2. Cautions and limitations:

- A. Not for use in atmospheres containing less than 19.5% oxygen.
- B. Not for use in atmospheres immediately dangerous to life or health.
- C. Do not exceed maximum use concentrations established by regulatory standards.
- J. Failure to properly use and maintain this product could result in injury or death.
- M. All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N. Never substitute, modify add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O. Refer to user's instructions and /or maintenance manuals for information on use and maintenance of these respirators.
- P. NIOSH dose not evaluate respirators for use as surgical masks.



Fitting Instructions:

- 1. Position the mask comfortably under your chin and against the bridge or your nose.
- 2. Stretch and pull the lower headband over the head and position below your ears. Stretch and pull the top headband on the back of your head above your ears.
- 3. USING BOTH HANDS, mold the metal nosepiece to the shape of your nose and cheek area. Using one hand to pinch the nosepiece may cause a bad fit: resulting in lower effective mask performance.
- 4. FIT-CHECK: Place both hands over the mask. Inhale sharply without disturbing the position of the mask. The mask should start to collapse. If not, or if air leaks around the edges, readjust mask until fit is sealed.

Warning:



Jiangyin Changhua Industrial Manufacturing Co.,Ltd.

Address: North Industrial Park, Huangtang, Xiake Town Jiangyin City, Jiangsu,China.
Tel: +86-510-86539881 Fax: +86-510-86913736
E-Mail:marketing@jychanghua.com Web:http://www.jychanghua.com

1. This product does not eliminate the risk of contracting any disease or infection.
2. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death.
3. A properly selected respirator is essential to protect your health. Before using this respirator, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use.
4. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19.5%.
5. Do not use when concentrations of contaminants are immediately dangerous to health or life.
6. Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs.
7. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator.
8. Never alter or modify this respirator in any way.
9. Only for single use. No maintenance necessary. Discard the used respirator after single use.
10. Keep respirators in the display box away from direct sunlight until use.
11. The user must be fit tested prior to use either qualitatively or quantitatively according to OSHA 1910.134.