MOLDEx®

Ideas that wear well.®

1712 N95

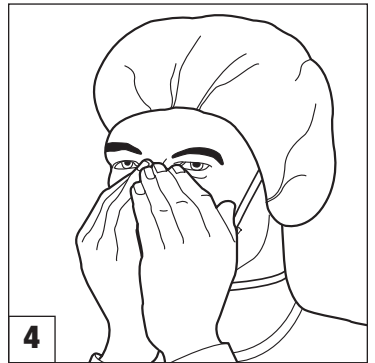
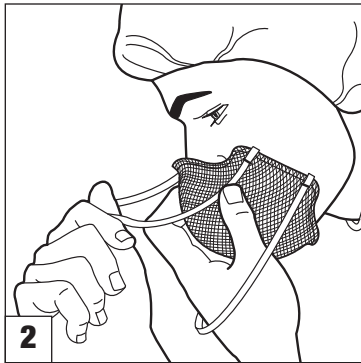
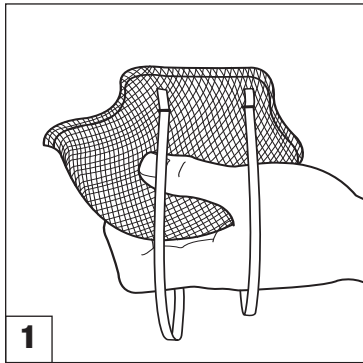
HEALTHCARE PARTICULATE RESPIRATOR & SURGICAL MASK

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FITTING

- If you cannot obtain a proper fit do not enter the contaminated area. OSHA regulation 29 CFR 1910.134 (f) requires that the user must be fit tested before being assigned a respirator. Use Moldex Qualitative Fit Test Kit or an OSHA approved Quantitative fit test such as TSI's PortaCount® with N95 Companion®
- Users must follow instructions each time respirator is worn. It's the wearer's responsibility to obtain a proper fit.
- To choose the correct model/size Moldex Respirator you must fit test and user seal check.

- **When changing from one Moldex model or size to another or when changing from another manufacturer's product to Moldex, you must fit test with the new respirator.**
- Not for use with beards, or other facial hair that prevents direct contact between the face and sealing surface of the respirator.
- Eyewear must not prevent direct contact between the face and sealing surface of the respirator.

1. Hold respirator in hand with molded nose contour (narrow end) at finger tips, allowing headstraps to fall below hand.
2. Place respirator under chin with molded nose contour (narrow end) up. Nose cushion must not be created inside respirator. Raise top strap to top back of head. Pull shorter bottom strap over head, below ears, to around neck. **Do not wear with only one strap because it may affect fit.**
3. Adjust respirator for comfortable fit.

4. Each time user enters contaminated area seal should be fit checked. Cover front of respirator by cupping both hands. **INHALE SHARPLY.** A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, adjust straps by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.

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WARNING

Failure to follow all instructions and warnings on the proper use of this respirator may reduce respirator effectiveness, wearer protection and may result in sickness or death. For additional information consult your supervisor, read all information on box, and phone +1 (800) 421-0668 or +1 (310) 837-6500, ext. 512/550.

LIMITED WARRANTY – IMPORTANT NOTICE TO PURCHASER
This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Moldex will be met by products sold. **Exclusive Remedies:** damages for the breach of this limited warranty are limited to the replacement of such quantity of Moldex products proved to be defectively manufactured. Except as provided above, Moldex shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.

Moldex Technical Service Department:
+1 (800) 421-0668 or +1 (310) 837-6500, ext. 512/5

Moldex and Ideas that wear well are registered trademarks. U.S. Patents Pending. The mesh design and the PVC-Free logo are trademarks of Moldex-Metric, Inc. U.S. Patents #D285,374 and #4,850,347

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WARNING TO USER:

1. Non-latex straps are used in the headbands of these respirators. However, individuals highly sensitive to natural rubber latex may have an allergic reaction.
2. **This product does not eliminate the risk of contracting any disease or infection.** Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards.
3. This product has not been sold with warning or use instructions for personnel involved in industrial or related situations. If you are considering such uses, first contact

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DESCRIPTION:

1. The Moldex Healthcare Series N95 Particulate Respirator and Surgical Mask is designed to help provide respiratory protection for the wearer. This product has been tested¹ and certified by NIOSH as an N95 respirator and as having a filter efficiency level of 95% or greater against particulate aerosols free of oil. It is fluid resistant,² disposable and may be worn in surgery or throughout the hospital.
2. This product contains no components made from natural rubber latex.

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MOLDEX[®]
Ideas that wear well.™

Printed on Recycled Paper

the Moldex Technical Services Department at +1 (800) 421-0668 or +1 (310) 837-6500, ext. 512/550.

RESTRICTIONS:

1. Before use, a written respiratory protection program in accordance with 29 CFR 1910.134 must be implemented.
2. This respirator does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen.
3. Prior to each use, carefully inspect the entire respirator, including filter media and strap attachment area for

Département des Services Techniques Moldex, au +1 (800) 421-0668 ou +1 (310) 837-6500, ext. 512/550.

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INTENDED USE:

The various models of Moldex Type N95 Healthcare Particulate Respirators and Surgical Masks meet CDC Guidelines for TB Exposure Control within healthcare facilities. These devices are also intended to be worn by healthcare personnel during surgical procedures to protect both the patient and healthcare personnel from the transfer of microorganisms, body fluids and particulate material.

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1712-710 REV A 2/10

- tears and damage. Staple or strap attachment perforations do not effect NIOSH approval.
4. If respirator is damaged, distorted, a proper fit cannot be obtained or breathing becomes difficult, leave contaminated area as soon as possible and replace respirator.
 5. If the respirator comes in contact with blood or fluids, including body fluids, leave contaminated area as soon as possible and discard and replace the respirator.
 6. Do not alter, modify or abuse this respirator.
 7. Store unused respirators in box/bag in a clean, dry, non-contaminated area.

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CONTRAINDICATIONS:

1. Not for use with beards, or other facial hair that prevents direct contact between the face and sealing surface of the respirator.
2. Eyewear must not prevent direct contact between the face and sealing surface of the respirator.
3. **Not** to be used on children.

¹ Tested in accordance with NIOSH 42 CFR 84
² Passed ASTM F 1862 @ 160mm Hg

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8. Dispose of respirator according to your employer's policy and local regulations.
9. Do not reuse or store for reuse or hang around neck unless your employer specifically authorizes reuse.
10. When used for surgical procedures, discard after every use.
11. Use respirator before the "use by" expiration date printed on box/bag.

For technical assistance call Moldex Technical Service Department, +1 (800) 421-0668 or +1 (310) 837-6500, ext. 512/550.

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