NPPTL COVID-19 Response: International Respirator Assessment

Manufacturer: Shenzhen Guidong Information Consulting Co., Ltd.

Model Tested: KN95 Folding Disposable Protective Mask

Date Tested: July 16, 2020

These findings pertain to the Shenzhen Guidong Information Consulting Co., Ltd., KN95 Folding Disposable Protective Mask. The packaging for this product indicates that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

Ten respirators were submitted for evaluation. The samples were tested using a modified version of NIOSH Standard Test Procedure (STP) TEB-APR-STP-0059. This modified assessment plan can be found here.

No certificate of approval was provided with the samples received; therefore, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 97.07% and 92.52%, respectively. Eight respirators measured more than 95%. Two respirators measured less than 95%.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

In addition, this product is an ear loop design. Currently, there are no NIOSH-approved products with ear loops; NIOSH-approved N95s have head bands. Furthermore, limited assessment of ear loop designs, indicate difficulty achieving a proper fit. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirators represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for <u>Crisis Capacity Strategies (during known</u> shortages).

Evaluation of International Respirators



Pictures have been added to the

end of this report.

Test: Modified TEB-APR-STP-0059

Date Tested: July 16, 2020

Report Prepared: July 16, 2020

Manufacturer: Shenzhen Guidong Information Consulting Co., Ltd.

Item Tested: KN95 Folding Disposable Protective Mask

Country of Certification: China (GB2626-2006)

Filter	Flow Rate (LPM)	Initial Filter Resistance (mmH ₂ O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency	
1	85	9.5	6.10	6.17	93.83	
2	85	12.8	3.35	3.35	96.65	
3	85	13.9	3.83	3.83	96.17	
4	85	17.9	3.51	3.51	96.49	
5	85	13.2	3.04	3.04	96.96	
6	85	12.0	2.39	2.93	97.07	
7	85	9.7	7.47	7.48	92.52	
8	85	9.5	4.38	4.38	95.62	
9	85	16.2	3.04	3.04	96.96	
10	85	8.6	3.02	3.02	96.98	
М	Minimum Filter Efficiency: 92.52			Maximum Filter Efficiency: 97.07		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of
 respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore
 cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.





[Notices]

- 1. Disposable supplies. Please dispose the mask after 4 hours' continuous use. Not re-usable.
- 2. Don't use a mask when its package is broken and please properly dispose the mask after use.
- 3. Please confirm the external packaging mark, production date & expiry date before use. Always use a mask within its validity period.
- 4. Please refer to the instructions before use and use it immediately after unpacked.
- 5. For those with cardiopulmonary insufficiency or any related symptoms, please seek your doctor's advice and use the masks carefully.
- 6. Don't use a mask for over 4 hours or frequently fold the nose metal strap.

(Brand Holder)

Fuvuan Leyu Medical Equipment (Zhongshan) Co., Ltd. [Address] Block D, No.7 Yongxin Street, Qunlian Village, Tanzhou Town, Zhongshan City [Tel] +86 0760-86726888

[Authorized Manufacturer] Shenzhen Guidong Information Consulting Co., Ltd [Factory Address] A2 factory of Zhonghan Electronics, Juyuan Industrial Zone, Qiaotang Road, Fuhai Street, Juyuan Industrict, Shenzhen City, Guangdong Province [Tel] +86 0755-89490113





NPPTL COVID-19 Response: International Respirator Assessment







