## NPPTL COVID-19 Response: International Respirator Assessment

Manufacturer: Dongguan Sure-on Industrial Ltd., (Makrite Industries, Inc.)

Model Tested: 9500-N95; Lot# 061520

Date Tested: October 6, 2020

These findings pertain to the Dongguan Sure-on Industrial Ltd., (Makrite Industries, Inc.), model 9500-N95; Lot# 061520. The packaging and labeling indicate that it is a NIOSH-approved product, under approval number TC-84A-5411. The packaging also indicates that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

The maximum and minimum filter efficiency was 98.95% and 94.00%, respectively. Nine respirators measured more than 95%. One respirator measured less than 95%.

This product has head bands/straps. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirators represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for <u>Crisis Capacity Strategies (during known shortages)</u>.

## **Evaluation of International Respirators**



Pictures have been added to the

end of this report.

**Test:** Modified TEB-APR-STP-0059

Date Tested: October 6, 2020

Report Prepared: October 7, 2020

Manufacturer: Dongguan Sure-on Industrial Ltd., (Makrite Industries, Inc.)

Item Tested: 9500-N95; Lot#061520


Country of Certification: China (GB2626), USA (claimed, 42 CFR 84)

Filter	Flow Rate (LPM)	Initial Filter Resistance (mmH₂O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency (%)
1	85	24.0	1.18	1.18	98.82
2	85	22.6	1.22	1.22	98.78
3	85	20.3	1.66	1.66	98.34
4	85	19.0	1.05	1.05	98.95
5	85	26.1	1.66	1.66	98.34
6	85	17.8	1.44	1.44	98.56
7	85	15.7	2.54	2.54	97.46
8	85	13.9	6.00	6.00	94.00
9	85	13.9	3.81	3.81	96.19
10	85	20.9	1.59	1.59	98.41
Minimum Filter Efficiency: 94.00% Maximum Filter Efficiency: 98.95%					98.95%

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of
  respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and
  therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.






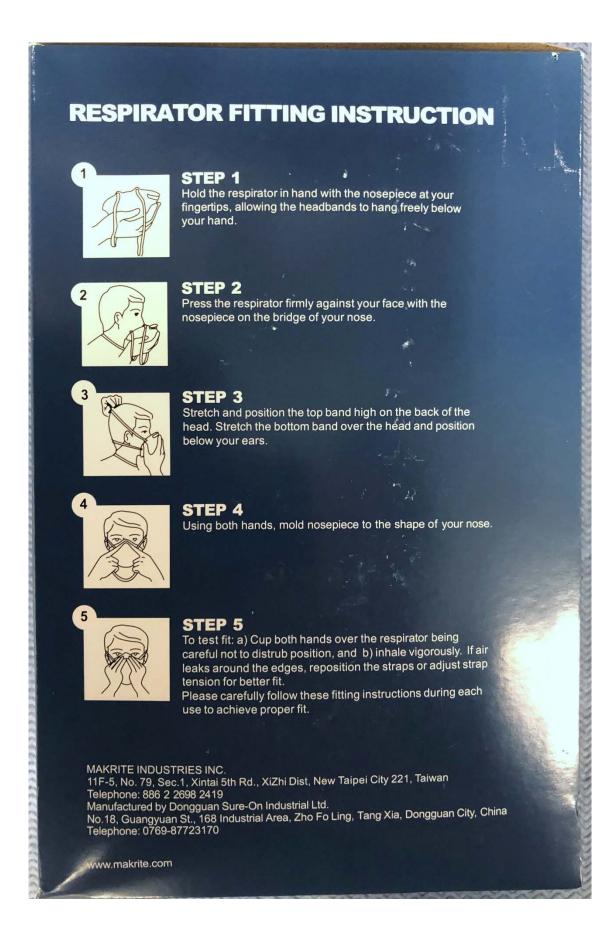





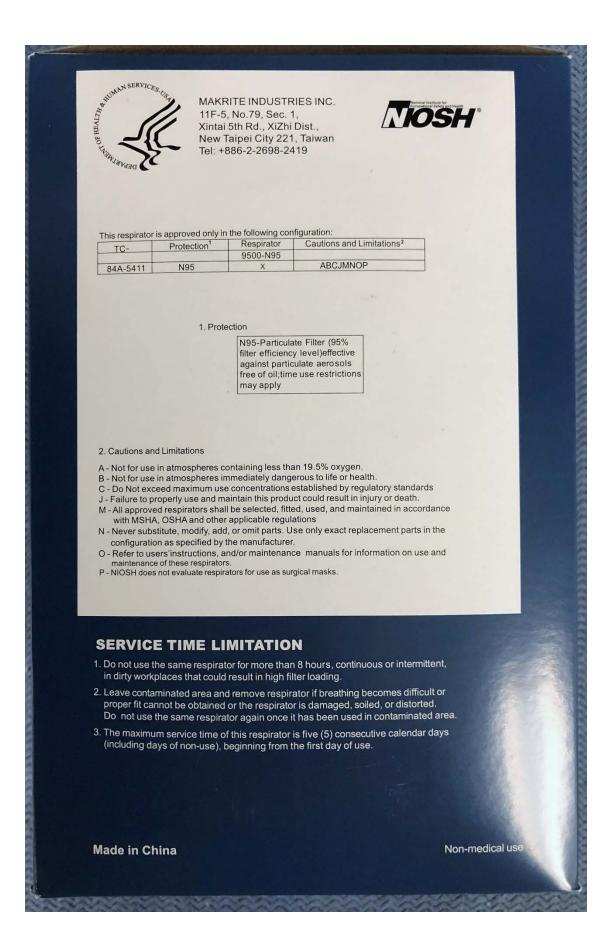







## NPPTL COVID-19 Response: International Respirator Assessment





## NPPTL COVID-19 Response: International Respirator Assessment

