Manufacturer: Suzhou Letian Protective Products Co., Ltd.

Model Tested: RRS-KN95-5PK Date Tested: November 19, 2020

These findings pertain to the Suzhou Letian Protective Products Co., Ltd., model RRS-KN95-5PK. The packaging and labeling for this product indicate that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

No certificate of approval was provided with the samples received; therefore, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 99.97% and 99.92%, respectively. All ten respirators measured more than 95%.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

In addition, this product is an ear loop design. Currently, there are no NIOSH-approved products with ear loops; NIOSH-approved N95s have head bands. Furthermore, limited assessment of ear loop designs, indicate difficulty achieving a proper fit. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirators represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for <u>Crisis Capacity Strategies (during known shortages)</u>.

Evaluation of International Respirators



Pictures have been added to the

end of this report.

Test: Modified TEB-APR-STP-0059

Date Tested: November 19, 2020

Report Prepared: November 19, 2020

Manufacturer: Suzhou Letian Protective Products Co., Ltd.

Item Tested: RRS-KN95-5PK


Country of Certification: China (GB2626-2006)

Filter	Flow Rate (LPM)	Initial Filter Resistance (mmH₂O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency (%)
1	85	20.3	0.04	0.04	99.96
2	85	19.6	0.06	0.07	99.93
3	85	15.6	0.03	0.03	99.97
4	85	17.3	0.04	0.04	99.96
5	85	18.9	0.08	0.08	99.92
6	85	24.1	0.03	0.03	99.97
7	85	20.2	0.03	0.03	99.97
8	85	22.5	0.07	0.07	99.93
9	85	19.4	0.03	0.03	99.97
10	85	22.2	0.03	0.03	99.97
Minimum Filter Efficiency: 99.92% Maximum Filter Efficiency: 99.97%					

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of
 respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and
 therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.





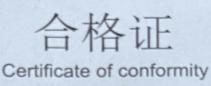

执行标准: GB2626-2006 KN95

Standard: GB2626-2006 KN95 产品名称: 非医用KN95口罩

Product: KN95 protective mask

品牌/型号: SUPPLYAID/RRS-KN95-5PK

Brand/Model: SUPPLYAID/RRS-KN95-5PK

参数: 190mm x 150mm, 5 个/袋

Specification: 100mm x 150mm 5 pcs / bag

产品材质: 无负债子, 8%+熔镀布的 18% 热风棉19.44%

Material: Nat-Noven fabric 52.78%+Meltblown2.223% Hot air cotton 19.44%

生产日期/批次: 20200725

Production Date: 20300725

保质期: 包裳木顶轨 果保里2年

Expire Date: The package is not unpacked and the shelf life is 2 year

生产商: 苏州乐天防护用品有限公司

Manufacturer: Suzhou Letian protective products Co.,Ltd

地址: 张家港市南丰镇南丰村

Address: Zhangjiagang City Nanfeng town Nanfeng village











