## NPPTL COVID-19 Response: International Respirator Assessment

Manufacturer: Gaomi City Hancheng Personal Protective Products Co., Ltd. Model Tested: Baokangjie 9001 Date Tested: April 22, 2020

These findings pertain to the Gaomi City Hancheng Personal Protective Products Co., Ltd., Baokangjie 9001. The labeling for this product indicates that it meets GB2626-2006KN95, the Chinese standard for Respiratory Protective Devices – Filtering Half Masks to Protect Against Particles – Requirements, Testing, Marking.

Ten respirators were submitted for evaluation. The samples were tested using a modified version of NIOSH Standard Test Procedure (STP) TEB-APR-STP-0059. This modified assessment plan can be found <u>here</u>.

No certificate of approval was provided with the samples received; therefore, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 99.15% and 98.19%, respectively. All ten respirators measured more than 95%.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

In addition, this product is an ear loop design. Currently, there are no NIOSH-approved products with ear loops; NIOSH-approved N95s have head bands. Furthermore, limited assessment of ear loop designs, indicate difficulty achieving a proper fit. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirator's represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for <u>Crisis Capacity Strategies (during known</u> <u>shortages)</u>.

## **Evaluation of International Respirators**

Test: Modified TEB-APR-STP-0059

Date Tested: April 22, 2020

Report Prepared: April 23, 2020

Manufacturer: Gaomi City Hancheng Personal Protective Products Co., Ltd.

Item Tested: Baokangjie 9001

Country of Certification: China (GB2626-2006KN95)

Filter	Flow Rate (Lpm)	Initial Filter Resistance (mmH <sub>2</sub> O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency	
1	85	12.7	1.23	1.26	98.74	
2	85	12.4	1.04	1.05	98.95	
3	85	13.1	0.85	0.85	99.15	
4	85	13.0	1.78	1.81	98.19	
5	85	12.7	1.14	1.14	98.86	
6	85	12.3	1.14	1.15	98.85	
7	85	13.6	1.24	1.24	98.76	
8	85	13.5	1.09	1.09	98.91	
9	85	13.0	0.92	0.92	99.08	
10	85	14.6	1.02	1.02	98.98	
	Minimum Filter Eff	iciency: 98.19	Maxim	Maximum Filter Efficiency: 99.15		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.




Pictures have been added to the end of this report.

1开前 鉴完好 1-2020-57.3 保康洁 自吸过滤式防颗粒物呼吸器 高效防粉尘、颗粒物、雾霾PM2.5 随弃式面罩(口罩)有呼吸阀 🗌 GB2626-2006 KN95 无呼吸阀 🗆 标准号:耳带式/头带式 保康洁 50/25 (防颗粒物口罩) 高密市瀚诚劳保用品有限公司





## NPPTL COVID-19 Response: International Respirator Assessment

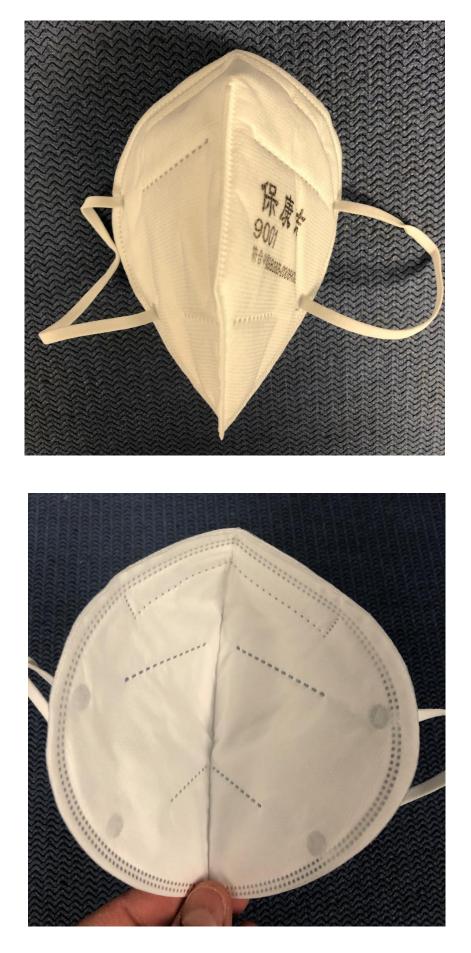

## NPPTL COVID-19 Response: International Respirator Assessment

