## NPPTL COVID-19 Response: International Respirator Assessment

Manufacturer: UNKNOWN

Model Tested: Kangsiluo Disposable KN95

Date Tested: May 1, 2020

These findings pertain to the respirators submitted for evaluation. The manufacturer is unknown. The model is the Kangsiluo Disposable KN95. The document submitted with this product indicates that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator). However, the standard appears to be mislabeled as GB2686-2006 KN9.

A certificate of approval was provided with the samples received; however, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 51.80% and 17.20%, respectively. All ten respirators measured less than 95%.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

In addition, this product is an ear loop design. Currently, there are no NIOSH-approved products with ear loops; NIOSH-approved N95s have head bands. Furthermore, limited assessment of ear loop designs indicate difficulty achieving a proper fit. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirator's represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for <u>Crisis Capacity Strategies (during known</u> shortages).

## **Evaluation of International Respirators**



Test: Modified TEB-APR-STP-0059

Date Tested: May 1, 2020

**Report Prepared:** May 3, 2020 **Report Updated:** May 11, 2020

Manufacturer: UNKNOWN

Item Tested: Kangsiluo Disposable KN95

Country of Certification: China (EN149:2001+A1:2009)


Pictures have been added to the end of this report.

Filter	Flow Rate (LPM)	Initial Filter Resistance (mmH <sub>2</sub> O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency
1	85	7.7	50.8	51.5	48.50
2	85	6.5	50.0	50.4	49.60
3	85	3.8	73.6	76.8	23.20
4	85	3.3	79.0	82.8	17.20
5	85	3.1	73.0	78.1	21.90
6	85	6.5	51.4	50.4	49.60
7	85	7.1	47.6	48.2	51.80
8	85	6.4	51.5	51.8	48.20
9	85	6.2	55.8	56.1	43.90
10	85	3.6	78.7	80.4	19.60
M	linimum Filter Effic	iency: 17.20	Maximum Filter Efficiency: 51.80		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of
  respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore
  cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control
  over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.

NPPTL COVID-19 Response: International Respirator Assessment



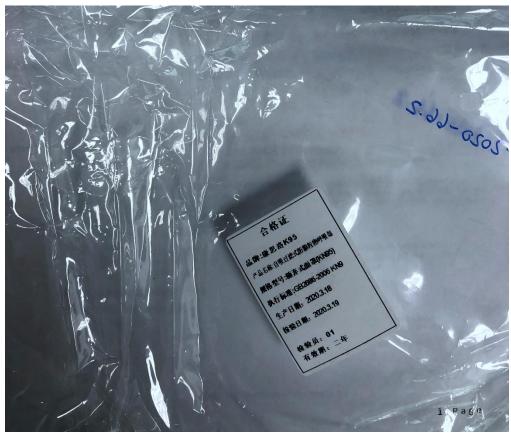


NPPTL COVID-19 Response: International Respirator Assessment





NPPTL COVID-19 Response: International Respirator Assessment



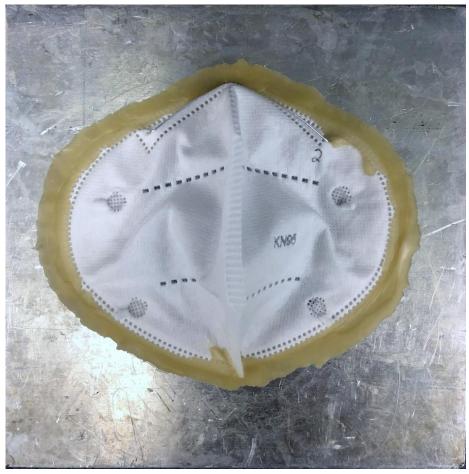

## NPPTL COVID-19 Response: International Respirator Assessment



