#### NPPTL COVID-19 Response: International Respirator Assessment

Manufacturer: Qingdao Hainuo Biological Engineering Co., Ltd.

Model Tested: C005

Date Tested: January 25, 2021

These findings pertain to the Qingdao Hainuo Biological Engineering Co., Ltd., model C005. The packaging for this product indicates that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

Ten respirators were submitted for evaluation. The samples were tested using a modified version of NIOSH Standard Test Procedure (STP) TEB-APR-STP-0059. This modified assessment plan can be found <a href="https://example.com/here">here</a>.

No certificate of approval was provided with the samples received; therefore, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 99.95% and 98.47%, respectively. All ten respirators measured more than 95% filter efficiency.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

In addition, this product is an ear loop design. Currently, there are no NIOSH-approved products with ear loops; NIOSH-approved N95s have head bands. Furthermore, limited assessment of ear loop designs, indicate difficulty achieving a proper fit. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirators represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for <u>Crisis Capacity Strategies (during known shortages)</u>.

## **Evaluation of International Respirators**



Pictures have been added to the

end of this report.

Test: Modified TEB-APR-STP-0059

Date Tested: January 25, 2021

Report Prepared: January 26, 2021

Manufacturer: Qingdao Hainuo Biological Engineering Co., Ltd.

Item Tested: C005

Country of Certification: China (GB2626-2006)

Filter	Flow Rate (LPM)	Initial Filter Resistance (mmH <sub>2</sub> O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency (%)	
1	85	19.7	0.05	0.05	99.95	
2	85	18.2	0.09	0.09	99.91	
3	85	19.1	0.06	0.06	99.94	
4	85	18.0	0.07	0.07	99.93	
5	85	17.1	0.07	0.07	99.93	
6	85	16.5	0.09	0.09	99.91	
7	85	15.6	1.44	1.58	98.42	
8	85	16.3	0.10	0.10	99.90	
9	85	17.5	0.11	0.11	99.89	
10	85	16.8	0.06	0.06	99.94	
ľ	Minimum Filter Efficiency: 98.47%			Maximum Filter Efficiency: 99.95%		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of
  respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and
  therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.

### NPPTL COVID-19 Response: International Respirator Assessment







# QUALITY INSPECTED 合格证

Product	KN95 Protective Face Mask			
产品名称	KN95防护口罩			
Model 款号	C005			
Specification 规格	20x8.5cm			
Ingredient 主要成分	61%non-woven fabric、39%melt blown 61%无纺布、39%熔喷布			
Standard 执行标准	GB2626-2006			
LOT No. 生产批号	201101			
MFG 生产日期	202011 QC 06			
DOE 有效期	5 years 5年 检验合格章			
EXP 检验员	06			
MFR 制造商	Qingdao Hainuo Biological Engineering Co., Ltd 青岛海诺生物工程有限公司			
ADD 地址	Jiangshan Industrial Park, Laixi, Qingdao, China 山东省青岛市莱西市姜山镇工业园			





## NPPTL COVID-19 Response: International Respirator Assessment





